

Please fill in all the gray information

Confirm eligibility
Vincent.Navarro@aphp.fr
Aurelie.Hanin@icm-institute.org

Patient study number

Sample collection

Please list any immunotherapy used before sample collection:

For all patients

Serum (1 red top tube, 6 mL)

Process in 2 hours or less

- Date and time of collection: .. / .. / ; .. :
- Centrifuge 1,500 g for 10 mn at room temperature
- Time of centrifugation: .. : ..
- Transfer supernatant into aliquots: 250 µL / aliquot; Number of aliquots
- Store at -80°C; time: .. : ..

Plasma (1 tube K2 EDTA, 6 mL)

Process in 2 hours or less

- Date and time of collection: .. / .. / ; .. :
- Centrifuge 1,500 g for 10 mn at room temperature
- Time of centrifugation: .. : ..
- Transfer supernatant into aliquots: 250 µL / aliquot; Number of aliquots
- Store at -80°C; time: .. : ..

If the family consents for genetic analyses

Whole blood (1 tube K2 EDTA, 6 mL)

- Date and time of collection: .. / .. / ; .. :
- Store at -80°C; time: .. : ..

If a lumbar puncture is done for clinical purposes

Cerebrospinal fluid (2-3 mL preferred, but all volumes are accepted)

Process in 2 hours or less

- Date and time of collection: .. / .. / ; .. :
- Centrifuge 1,500 g for 10 mn at room temperature
- Time of centrifugation: .. :
- Transfer supernatant into aliquots: 250 µL / aliquot; Number of aliquots
- Store at -80°C; time: .. :

Samples will be picked up by the carrier AreaTime

Contact: aurelie.hanin@icm-institute.org; zineb.hayatou@icm-institute.org; lea.cosme-ext@aphp.fr